



DANALOGIC

i-FIT

INTELLIGENT FITTING

# Aventa 2.9 installation

A step by step guide

# Aventa 2.9 Installation Guide

We recommend that any other older version of Aventa is removed (using the control panel: Add/remove programmes or the Aventa un-install tool available on the CD) prior to installing Aventa 2.9.



When installing Aventa 2.9 on a machine with Auditbase you must open Auditbase first and minimise it. Close all other programs before commencing the installation.

## 1: Click: setup.exe

Open CD contents and double click on the icon setup (may either appear as **'setup.exe'** or **'setup'**)

## 2: Choose: Setup Language

Chose **English (United States)** then click OK

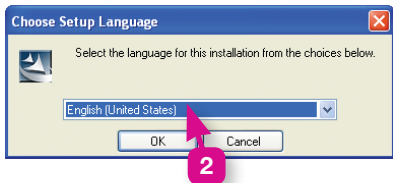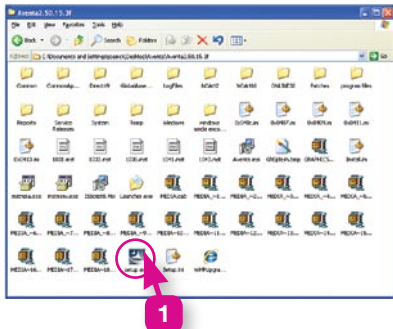


# Aventa 2.9 Installation Guide

## 3: Click Next

## 4: Select: Market Selection

Select the country as Great Britain and then click **Next**.

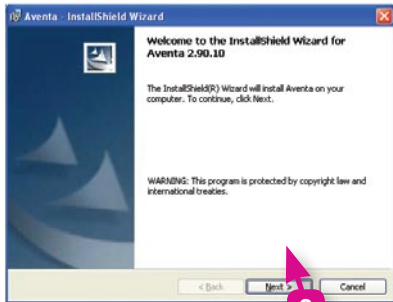

# Aventa 2.9 Installation Guide

## 5: License Agreement

Put a dot in the radio button to accept the licence and click **Next**.

## 6: Select: Destination Folder

Click **Next**.




## Aventa 2.9 Installation Guide

### 7: Select Program Features

Click on the red cross next to the word **'English'**, select **'This feature and all subfeatures'** and click **Next**.

### 8: Installation

Click **Install**.

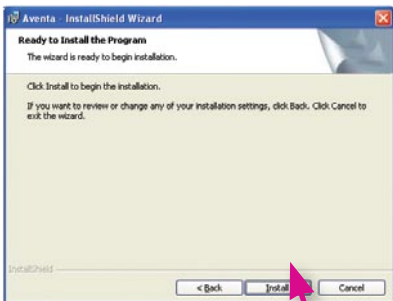



## Aventa 2.9 Installation Guide

9: Wait for Aventa to finish installing files to your computer.

### 10: To exit Installation

Click on **Finish** to exit the wizard.

### 11: Restart your computer

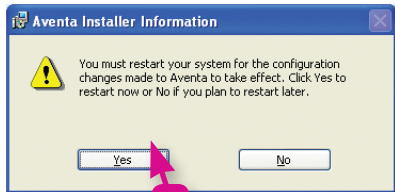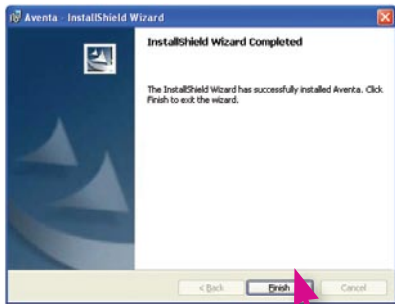



## Aventa: System requirements

### **Operating system:**

Windows 2000 SP2 or higher  
Or Windows XP SP2 or higher  
Or Windows Vista  
Does not support 64-bit

### **Display:**

Minimum screen resolution of  
1024x768 pixels

### **Disk space on hard drive:**

Minimum 1GB  
Recommended 2GB

### **CPU speed:**

Minimum Pentium III, 800 MHz  
Recommended Pentium IV, 1 GHz

### **Memory:**

Minimum 256MB RAM  
Recommended 512MB RAM